Instructions For Use  
Veiledning  
Gebrauchsanweisung  
Indicaciones de uso  
Käyttöohjeet  
Mode d'emploi  
Istruzioni per l'uso  
Gebruiksaanwijzing  
Bruksanvisning

Renew Medical, Inc.  
1065 E. Hillsdale Blvd  
Suite 410  
Foster City, CA 94404 USA

**EC REP** EMERGO EUROPE  
Prinsessegracht 20  
2514 AP, The Hague  
The Netherlands

LAB675 Rev B, 2018-08-29

USA: 888.987.2929  
support@renew-medical.com  
www.renew-medical.com

UK: 0800 542 0814  
support@renew-medical.uk  
www.renew-medical.uk

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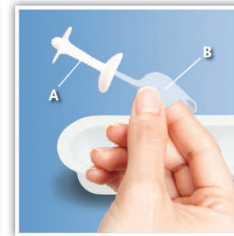
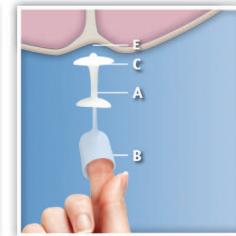
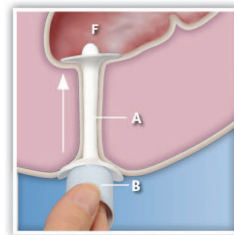
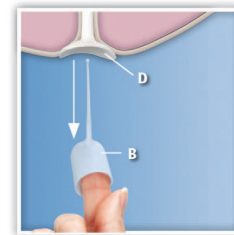
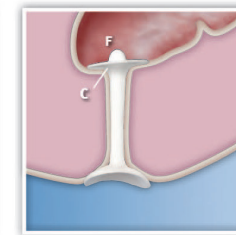
6.

## INDICATIONS FOR USE

The Renew® Insert is indicated for the management of Accidental Bowel Leakage (ABL) due to bowel incontinence. The Renew Insert is designed for self-insertion to seal and help prevent the involuntary leakage of stool from the rectum.

## DESCRIPTION

Renew Inserts are made of soft silicone and are safe and comfortable to use in the rectum. Renew Inserts and the removable applicator are both single-use and non-sterile. Renew Inserts are available in 2 sizes: Regular and Large.

## Possible Side Effects May Include:

False urge to empty your bowels.  
Mild irritation of anal area.

## Warnings and Precautions

Bowel incontinence may be medically treated.

The Renew Insert is not intended to replace surgical options to treat ABL.

Consult your doctor for professional advice.

Do not use if you have an allergy to silicone.  
Do not use with suppositories.

Discontinue using the Renew Inserts and consult your doctor if you experience the following:

- Bleeding hemorrhoids
- Rectal bleeding or bloody stool
- Anal/rectal pain, irritation, spasms, fissure or tearing
- Undergone rectal surgery within the past 4 weeks
- Displacement of the Insert upward into the anus and it is not expelled with the next bowel movement

Do not use if opened or damaged. Single use only. Non-sterile.

## US CLINICAL TRIAL SUMMARY

A clinical study has shown that Renew Inserts are safe and work as indicated. Three sites enrolled 97 subjects in this study. Safety data were analyzed on 91 treated subjects who used Renew Inserts. Reduction in the frequency of Accidental Bowel Leakage (ABL) episodes was measured by comparing pre- and post-treatment results on 85 subjects who used the Renew Insert for at least 1 week or more during the 12-week Treatment Period. Reduction in the Wexner Score (an assessment of bowel incontinence) was analyzed on 77 subjects who had both a pre- and post-treatment Wexner score.

## Benefits

The clinical study demonstrated that the Renew Insert helps prevent Accidental Bowel Leakage, delivering the following benefits:

- A median reduction of 81.8% in the frequency of ABL episodes
- Over 91% of the Renew Insert users demonstrated a  $\geq 20\%$  reduction in ABL episodes
- A median reduction of 29.4% in the Wexner score
- Overall satisfaction among the majority of Insert users was high, with 91.4% rating the overall experience and ease of use as a 9.5 (median) on a 10 point scale

## Risks

There were no reported severe, serious or unanticipated adverse events. Almost all (98.7%) of the adverse events possibly related to the Renew Insert were rated as 'mild' and included the sensation of urge, irritation or soreness around the anal area.

Displacement of the Insert may occur when the lower (bottom) disk of the Insert becomes temporarily displaced and moves upward into the anus. Displacements of the Insert were reported by 23.9% of the study participants. Instances of displacement of the Insert were resolved predominately by natural expulsion during subsequent bowel movements.

## FREQUENTLY ASKED QUESTIONS

## 1. Have Renew Inserts been clinically tested?

Yes. Clinical studies in the USA have demonstrated that the Renew Inserts are safe and reduce Accidental Bowel Leakage by 81.8%\*.

## 2. Are Renew Inserts comfortable to insert and wear?

Yes. Clinical studies show Renew Inserts are: soft, fit the contours of the body, comfortable to wear and easy to use\*. Inserting a Renew Insert properly will not cause discomfort. Just follow the step-by-step instructions. If you still experience discomfort, discontinue use.

\*U.S. Clinical Study (Renew 210 CLD) published in Dis Colon Rectum 2015; 58: 892-898.

## 3. How do I know when the Renew Insert is properly in place?

When the Renew Insert is properly in place, the top disk is resting just inside the rectum and the bottom disk is just outside the anus. You should not feel any discomfort.

## 4. How long can I wear a Renew Insert?

Renew Inserts may be worn continuously, day and night. A single Renew Insert can be worn until your next bowel movement, or changed as desired. After one is expelled, you may replace it with a new Insert. Never use more than one Insert at a time.

### 5. What are some insertion positions?

You may want to sit on the toilet and insert from the front or back, or lie down on a bed. You should use whichever insertion position is most comfortable for you. Avoid multiple insertion attempts to limit the exposure of stool to other parts of the body; use a new Insert if this happens.

### 6. Can a Renew Insert get lost inside my body?

No. Clinical research has shown that Renew Inserts come out with a person's next bowel movement\*. The Renew Insert is designed so the top and bottom disks hold it in place in the anal canal between the rectum and anus. In the unlikely event a Renew Insert moves into the anus, it will be naturally expelled with your next bowel movement.

### 7. What if the Renew Insert comes out when I go to the toilet and pass wind (gas) or urinate?

If this happens, simply replace it with a new Renew Insert. If you are using a Regular size Renew Insert and this happens frequently, you may want to try the Large size Renew Insert.

### 8. Will I be able to pass wind (gas) with the Renew Insert in place?

Yes. The Renew Insert still allows for the passage of wind (gas). If you are using a Large size Renew Insert and feel discomfort from wind (gas), try a Regular size Renew Insert.

### 9. Can I bathe and be physically active when I'm wearing my Renew Insert?

Yes. You can do all of your activities while using a Renew Insert.

### 10. Should I flush the applicator and Renew Insert down the toilet?

Flushing of the applicator and the Renew Insert is not recommended. Dispose of the used Renew Insert in household waste.

## SIZING

Renew Inserts come in two sizes, Regular and Large. Start with the Regular size Renew Insert. If the Regular size comes out at unexpected times or if you experience leakage, try the Large size Renew Insert.

## PREPARATION

**Step 1.** Do not use a Renew Insert package unless it is fully sealed. Wash your hands. Peel back the top of the package to fully expose the Renew Insert.

**Step 2.** Place your index finger into the light blue removable applicator and lift the Insert out of its package.

**Step 3.** Sit relaxed on a toilet and insert from the front or back. Other common insertion positions are to raise yourself up onto one hip (while sitting on the toilet) or lie down on a bed (if this is more comfortable). Position the top disc of the Renew Insert so it's lightly resting on the anus. Note: If the Insert falls off the applicator prior to insertion, discard it and use a new one.

## INSERTION

**Step 4.** Relax your muscles and gently slide the Renew Insert into the anus until you feel the applicator touch the anus.

**Step 5.** Gently withdraw the applicator. The bottom disk of the Renew Insert should be resting just outside the anus.

**Step 6.** Discard the applicator in a waste receptacle. Wash your hands after use. The Renew Insert is now properly in place and sealing from the inside to help prevent ABL.

## REMOVAL

**Step 7.** Simply pull on the bottom disk outside the anus to remove the Renew Insert and dispose of it in household waste. Wash your hands after direct contact with the used Insert.

## REPLACEMENT

**Step 8.** For optimal protection, use a new Renew Insert after each bowel movement.

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